



Sample Collection for Oncotype DX® AR-V7 Nucleus Detect™ Test

PREPARE

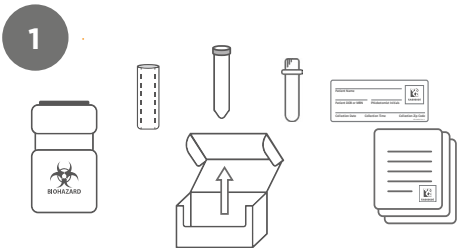

1 Open the collection kit to access the blood collection tube.

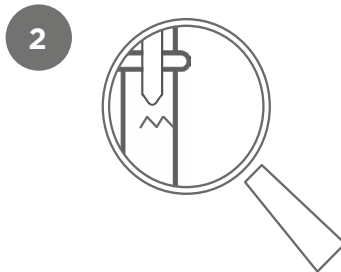


2 Inspect for broken or expired items. DO NOT REMOVE THERMAL PACKS OR DISCARD ANY ITEMS




3 Complete the AR-V7 Nucleus Detect test order form, fill out the blood tube label and apply all (3) phlebotomist labels as specified.


COLLECT

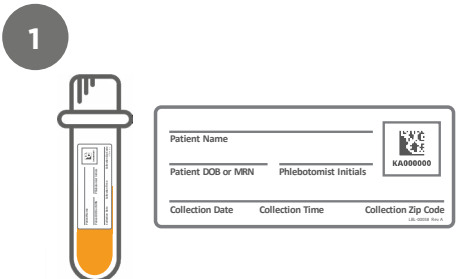


1 Label the tube with the patient's first and last name, DOB, and/or MRN, and date/time of specimen collection (Month, Day and Year).

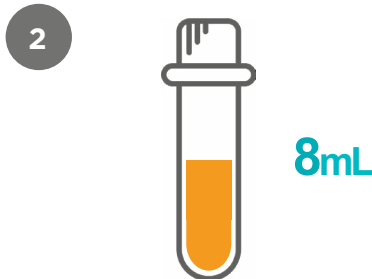


2 Using the included blood collection tube, collect at least 8ml peripheral blood.

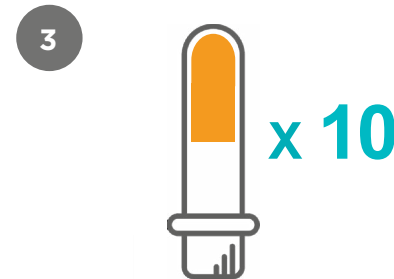


3 Immediately INVERT the tube ten (10x) times after blood draw to prevent clotting.

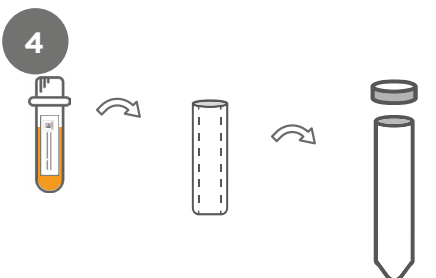


4 Place the blood collection tube into absorbent sleeve and conical.

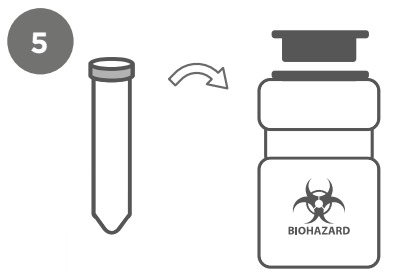

5 Replace the conical cap. Place the conical into the center of the canister and replace the canister cap.

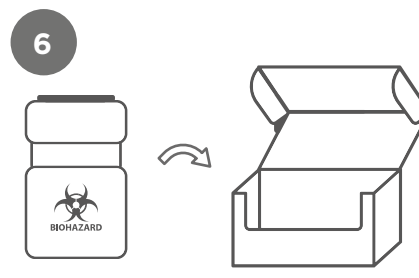

6 Place the canister into the collection kit box.

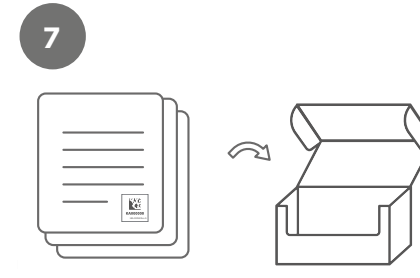


7 Fold and place the completed order form, a copy of the patient's insurance, and any other accompanying documents into the kit box.

Sample Collection for Oncotype DX® AR-V7 Nucleus Detect™ Test


SHIP

1

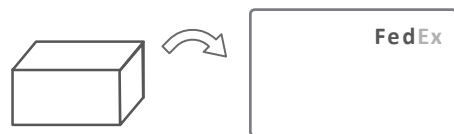


Place the collection kit box directly into the pre-addressed FedEx Clinical Pak, seal the Pak, and STORE at ROOM TEMPERATURE (15-30 degrees Celsius) pending shipment.

2

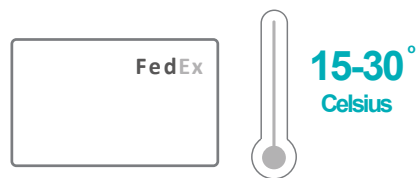


Place the collection kit box directly into the pre-addressed FedEx Clinical Pak, seal the Pak, and STORE at ROOM TEMPERATURE (15-30 degrees Celsius) pending shipment.

3

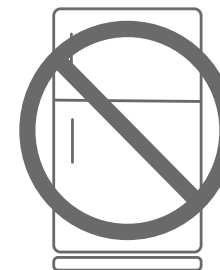


DO NOT REFRIGERATE OR FREEZE KIT OR CONTENTS

4

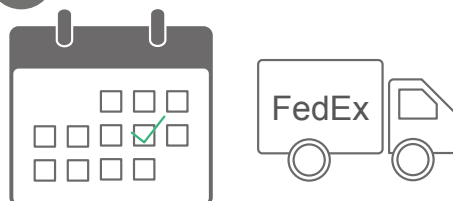

ENSURE FEDEX PICKUP is scheduled for the SAME DAY sample is collected.

5

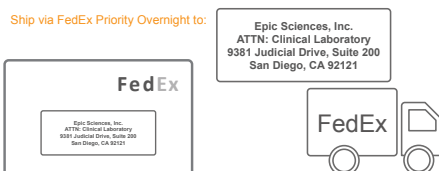

Ship via FedEx Priority Overnight to:
Epic Sciences, Inc.
ATTN: Clinical Laboratory
9381 Judicial Drive, Suite 200
San Diego, CA 92121

Customer Service:
Genomic Health, Inc.
1-866-ONCOTYPE
(1-866-662-6897)
www.oncotypedx.com/contact



AR-V7 Nucleus Detect, Genomic Health, Oncotype IQ, and Oncotype DX are registered trademarks of Genomic Health, Inc. © 2018 Co-owned by Epic Sciences, Inc. and Genomic Health, Inc. All rights reserved. GHI60019_0518

oncotypeDX®
AR-V7 Nucleus Detect
powered by Epic Sciences

oncotypeIQ
Genomic Intelligence Platform

Genomic Health
LIFE. CHANGING.